

CLINITEK Status Connect System

Transmit a wealth of clinical information with a connected urinalysis solution

Eliminate subjectivity with automated testing at the point of care

siemens-healthineers.com/poc



The CLINITEK Status Connect System: Trusted urinalysis testing at the point of care

Urinalysis provides a wealth of indispensable clinical information. Performance-driven urinalysis solutions from Siemens Healthineers support early detection, monitoring, and management of diseases and conditions, including diabetes, kidney disease, urinary tract infections (UTIs), and pregnancy, enabling you to deliver results with confidence.

The CLINITEK Status® Connect System with Auto-Checks® Technology simplifies and streamlines your daily operations. This automated point-of-care urinalysis and hCG pregnancy testing solution eliminates the subjectivity of a visual read, helps improve accuracy, saves time, and allows for increased productivity.

Automated analyzer helps improve clinical outcomes

- Eliminates the subjectivity of visually read tests and delivers actionable results
- Printed/automatically transmitted results save time and minimize transcription errors
- Automatic calculation of albumin-to-creatinine and protein-to-creatinine ratios for early kidney disease detection reduces the need for 24-hour urine collection

Intuitive, easy-to-use touch screen interface is ideal for point-of-care settings

- Convenient entry of patient/operator information
- On-screen display of complete patient results, available when you need them
- Streamlines workflow and eliminates the need for manual documentation
- Simple, efficient operation delivers urine strip results in just 1 minute and hCG results in 2–5 minutes

Next generation CLINITEK Status Connect platform provides flexible data management and connectivity

- Transmits results to the electronic medical record (EMR), LIS, or third-party POC data manager
- On-board Wi-Fi with expanded network compatibility†
- Bi-directional connectivity enables remote configuration
- Advanced QC management for tracking of QC results
- Connects to Siemens Healthineers POCcelerator™ Data Management System



Simplifies your testing process and improves risk management

- Barcode of patient/operator ID and reagent information reduces transcription errors
- Patient/operator ID ensures accurate audit trail
- Advanced operator and QC management allows you to drive compliance
 - operator lockout prevents unauthorized use and protects information
 - QC lockout prevents the reporting of patient results if process is not followed

†Feature not available in all countries. Please contact your local representative for availability.



The broadest menu for point-of-care urinalysis testing on one analyzer

Multistix® Family of Reagent Strips	Most widely used urinalysis strips for routine testing
CLINITEK® Microalbumin 2 Reagent Strips	Albumin-to-Creatinine ratio test to detect early kidney disease for patients with confirmed diabetes
CLINITEK® hCG Pregnancy Test	Qualitative hCG test to quickly identify or exclude pregnancy



Internal Quality Control

Chek-Stix® Positive and Negative Control Strips can be used to verify the integrity of results, providing a convenient basis for a urinalysis quality control program.



Automated urinalysis solutions to fit your needs

Feature/Function	CLINITEK Status Connect System	CLINITEK Status+ Analyzer
Auto-Checks Technology <ul style="list-style-type: none"> • Detect humidity exposure • Check for common sample interferences[‡] • Identify strip type 	• • •	• • •
Quality Control Management <ul style="list-style-type: none"> • QC scheduling • QC lockout • Transmission of QC results 	• • •	— — —
Operator Management <ul style="list-style-type: none"> • Operator lockout • Operator list download 	• •	• —
Barcode patient/operator IDs, lot information	•	—
Remote configuration management	•	—
Connection type	Ethernet—bi-directional	Serial—uni-directional


[‡]Feature not available in all countries.



At Siemens Healthineers, we constantly look for ways to help our customers reduce their environmental burden. The CLINITEK Status Connect System is built primarily from plastics and metals that can be recycled almost completely for materials or energy. This connected solution allows our customers to go paperless.

Feel confident in your results

Siemens Healthineers proprietary Auto-Checks Technology helps to ensure the quality and accuracy of data while facilitating an enhanced interpretation of results. Our testing solutions provide important automated quality checks with every test.

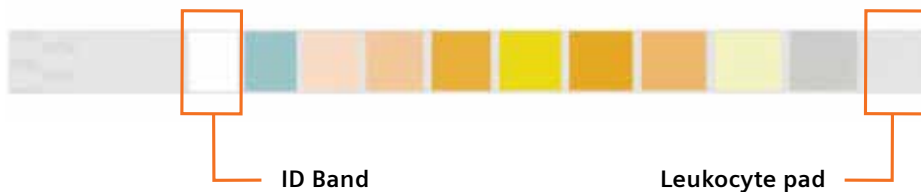

Three Functionalities of Auto-checks

1 Identify strip type

Our CLINITEK family of urine analyzers have software that recognizes the ID bands and automatically identifies strip type; no manual step is needed to adjust to a different type.

Other analyzers cannot automatically detect strip types requiring additional manual steps to the workflow, and potentially leading to wrong results if other brand strips are run on the analyzer.

2 Detect humidity exposure

Uses leukocyte pad to detect strip humidity overexposure.* Does not report results if overly exposed, minimizing false positives.

- Algorithms designed to detect discoloration on the Leukocyte pad due to humidity exposure
- Once detected, error code E69 will be reported prompting users to use a new strip

3 Check for common sample interferences†

Detects common sample interferences and informs the healthcare professional of a potentially compromised result.



*Available only when using Siemens Healthineers test strips with ID or color bands.

†Feature not available in all countries. Please contact your local representative for availability.

At Siemens Healthineers, our purpose is to enable healthcare providers to increase value by empowering them on their journey toward expanding precision medicine, transforming care delivery, and improving patient experience, all made possible by digitalizing healthcare.

An estimated 5 million patients globally benefit every day from our innovative technologies and services in the areas of diagnostic and therapeutic imaging, laboratory diagnostics, and molecular medicine, as well as digital health and enterprise services.

We are a leading medical technology company with over 120 years of experience and 18,000 patents globally. Through the dedication of more than 50,000 colleagues in 75 countries, we will continue to innovate and shape the future of healthcare.

Auto-Checks, Chek-Stix, CLINITEK, CLINITEK Status, CLINITEST, Multistix, POCcelerator, Point of Care Ecosystem, and all associated marks are trademarks of Siemens Healthcare Diagnostics Inc., or its affiliates. All other trademarks and brands are the property of their respective owners.

Product availability may vary from country to country and is subject to varying regulatory requirements. Please contact your local representative for availability.

Siemens Healthineers Headquarters

Siemens Healthcare GmbH
Henkestr. 127
91052 Erlangen, Germany
Phone: +49 9131 84-0
siemens-healthineers.com

Published by

Siemens Healthcare Diagnostics Inc.
Point of Care Diagnostics
2 Edgewater Drive
Norwood, MA 02062-4637
USA
Phone: +1 781-269-3000